



# SELECTIVE MICRO® CLEAN 2L100A

PRODUCES TWO LITERS OF 100 PPM CHLORINE  
DIOXIDE SOLUTION WHEN ACTIVATED

**KEEP OUT OF REACH OF CHILDREN**

**DANGER**

(See back panel for other cautions)

Active Ingredient: Sodium Chlorite (CAS No. 7758-19-2).....	25.5%
Inert Ingredients:.....	74.5%
Total:.....	100.0%
Net Weight of contents.....	0.10 ounces (2.86 grams)

Amount of Chlorine Dioxide generated = 0.01%

## THIS SIDE UP AFTER FILLING

**Intended Use:** To clean and remove organic matter from fluid lines and reservoirs in clinical analyzers. To clean and remove organic matter in holding tanks, water reservoirs, and distributions lines in water systems and water purification systems.



**DANGER:** MAY INTENSIFY FIRE; OXIDIZER.  
CAUSES SEVERE SKIN BURNS AND EYE DAMAGE.

**PRECAUTIONARY STATEMENTS:**  
AVOID BREATHING FUMES. AVOID CONTACT WITH SKIN AND EYES. WEAR SUITABLE PROTECTIVE CLOTHING, GLOVES, AND EYE/FACE PROTECTION. DO NOT BREATHE CHLORINE DIOXIDE FUMES, USE IN WELL VENTILATED AREA.

*NOTE: GHS HAZARD STATEMENTS APPLY TO THE DRY CHEMICAL. THIS PRODUCT FULLY COMPARTMENTALIZES DRY INGREDIENTS INSIDE A HYDROPHOBIC ENVELOPE. UNDER NORMAL USE CONDITIONS AND UNLESS SEALED PRODUCT ENVELOPE IS COMPROMISED, CONTACT WITH THE DRY INGREDIENTS WILL NOT OCCUR. READ SDS BEFORE USE.*

## DIRECTIONS FOR USE:

DO NOT USE THIS PRODUCT IN A MANNER INCONSISTENT WITH THE LABELING.

1. Fill the 2L100A pouch with two (2) liters of distilled or deionized water.
2. Lay filled pouch so that the Selective Micro®Clean label is in full view on top.
3. WAIT 1 HOUR BEFORE USE TO ENSURE FULL STRENGTH.
4. Shake gently before use.
5. Direct spout away from face when opening, work in a well-ventilated area, and avoid inhaling fumes.
6. Follow specific instrument cleaning protocol to complete procedure.
7. Store unused solution (if any) in pouch in refrigerator or cool place out of direct sunlight. Use solution within 72 hours of generation.
8. Activate prior to expiration date stamped on pouch.

Use in conjunction with *Selective Micro® Chlorine Dioxide Test Strips*.

## WARRANTY STATEMENT:

The Company warrants the product to be free from defects in material and workmanship. This product does not make antimicrobial claims. THE COMPANY MAKES NO WARRANTY THAT THE GOODS SHALL BE MERCHANTABILITY. THE COMPANY MAKES NO WARRANTY, EXPRESS OR IMPLIED, EXCEPT SUCH AS IS EXPRESSLY SET FORTH HEREIN. The Company shall not be liable for any incidental or consequential damages for any breach of warranty. The Company's liability for any breach of warranty shall be limited to the purchase price of the product.

Manufactured for: Selective Micro Technologies  
6200 Avery Rd. – Suite A  
Dublin, OH 43016